ACCU-CHEK® Solo

CANNULA ASSEMBLY & PUMP HOLDER INSTRUCTIONS FOR USE

Roche

ACCU-CHEK and ACCU-CHEK SOLO are trademarks of Roche.

© 2022 Roche Diabetes Care

Roche Diabetes Care GmbH
Sandhofer Strasse 116
68305 Mannheim, Germany
www.accu-check.com

Last update: 2022-03

EN Instructions for use

Accu-Chek Solo Cannula Assembly & Pump Holder

Intended Use

The Accu-Chek Solo cannula assembly consists of the cannula casing and the steriley cannula. It creates a connection between the micropump and the body to channel the insulin into the body. The Accu-Chek Solo pump holder is a plate that is adhered to the skin to fix the cannula in place. It also holds the Accu-Chek Solo micropump in place.

Package Contents and Storage Conditions

See outer packaging. You can find the symbol explanations in the User's Manual of the Accu-Chek Solo micropump system.

Component Overview

Pump holder:

- A Cannula opening with cannula support
- B Flap for detaching the micropump
- C Adhesive pad
- D Hooks for attaching the micropump

Cannula assembly:

- E Cannula casing with handle
- F Cannula head (grey)
- G Cannula
- H Introducer needle

Pump holder and cannula form the infusion assembly of the Accu-Chek Solo micropump system.

Do not touch the cannula (G) or the introducer needle (H) before insertion.

This product contains pointed and sharp-edged parts that may cause injury. Keep pointed and sharp-edged parts away from small children and vulnerable persons.

WARNING

Risk of suffocation

This product contains small parts that may be swallowed. Keep small parts away from small children and vulnerable persons.

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Read these instructions for use and the instructions in the User's Manual of the Accu-Chek Solo micropump system. Take particular note of all warnings, information and notes required for the safe and proper use of the micropump system.

This is a sterile, single use product that has been sterilised with ethylene oxide. You can use the infusion assembly for up to 3 days.

Before attaching the infusion assembly, choose a suitable infusion site. Sites with sufficient subcutaneous tissue are the most suitable. If required, ask your healthcare team for advice.

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Attaching the Infusion Assembly

To attach the infusion assembly, you need an Accu-Chek Solo insertion device, an Accu-Chek Solo cannula assembly & pump holder, wipes, and disinfectant.

To replace a component of the micropump system, always use the Replace menu on the diabetes manager.

To view an animated video of the handling steps, tap Help on the diabetes manager.

1 Wash your hands and dry them thoroughly. Disinfect the infusion site on your body according to the instructions given by your healthcare professional. Make sure that the infusion site is dry and free from residues.

2 Attach the hook on the pump holder (D) to the underside of the insertion device. Press the pump holder onto the underside of the insertion device until it locks into place. Check whether the pump holder is correctly locked into place.

3 Prime the insertion device by rotating the priming handle clockwise in the direction of the arrow, as far as it will go. When the insertion device is fully primed, you will hear a click.

4 Insert the cannula assembly into the cannula assembly slot. Push the cannula assembly into the slot until you hear it click into place. Use the positioning aid to check whether the cannula assembly is in the correct position.

5 Remove both parts of the protective film from the adhesive pad (C).

6 Holding the skin taut, press the insertion device firmly against the selected site on the body.

7 Press the blue release button to insert the cannula under the skin.

8 Smooth over the adhesive pad (A) around the infusion assembly so that the infusion assembly is in good contact with the skin.

9 Press the detach button and detach the insertion device from the infusion assembly. If possible, keep the adhesive pad in position with the other hand.

10 Press the infusion assembly and the edges of the adhesive pad against the skin so that the adhesive pad is smooth on the skin. Check whether the grey cannula head (F) in the cannula opening (A) is visible and flush with the opening. If this is not the case, repeat steps 1 to 10 using a new pump holder and a new cannula assembly.

Check the infusion site regularly. If the infusion site appears inflamed or has become contaminated, immediately replace the infusion assembly and choose another infusion site.

Removing the Infusion Assembly

To loosen the adhesive pad, you may apply warm water or an oily ointment on the adhesive pad.

I Press the flap to detach the micropump (B) and remove the pump from the infusion assembly.

II Loosen the edges of the adhesive pad (C) and pull the infusion assembly off to the center.

Disposal

WARNING

Risk of infection

Used components carry a risk of infection.

Dispose of the cannula assembly safely, in such a way that the introducer needle does not cause injury to anyone.

Dispose of used products according to local regulations. For more information about correct disposal, contact your local council or authority.

Distributed in the United Kingdom by:

Roche Diabetes Care Limited
Chipping Barnet, London
West Sussex, RH15 8NT, United Kingdom

Accu-Chek Pump Carline: 11¹

UK Freephone number: 0800 731 22 91

11¹calls may be recorded for training purposes.

Some mobile operators may charge for calls to these numbers.

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