## Instructions for use

### Accu-Chek Solo Reservoir Assembly

#### Intended Use

The Accu-Chek Solo reservoir assembly (including the filling aid and the handle for the piston rod) is a part of the micropump in addition to the Accu-Chek Solo pump base. The reservoir is a sterile container for holding the insulin that the micropump delivers to the body.

#### Package Contents and Storage Conditions

See outer packaging. You can find the symbol explanations in the User's Manual of the Accu-Chek Solo micropump system.

#### Component Overview

- A Reservoir
- B Filling aid (blue)
- C Handle for the piston rod (blue)
- D Protective film for battery (orange)
- E Reservoir needle
- F Piston rod

Do not touch the needle of the filling aid (B) or the reservoir needle (E).

This product contains pointed and sharp-edged parts that may cause injury. Keep pointed and sharp-edged parts away from small children and vulnerable persons.

#### WARNING

##### Risk of suffocation

This product contains small parts that may be swallowed. Keep small parts away from small children and vulnerable persons.

#### Before You Get Started

Read these instructions for use and the instructions in the User's Manual of the Accu-Chek Solo micropump system. Take particular note of all warnings, information and notes required for the safe and proper use of the micropump system.

This is a sterile, single use product that has been sterilised with ethylene oxide. You can use the reservoir assembly for up to 4 days.

Replace the reservoir immediately if it has been exposed to mechanical stress (for example, if it was hit or dropped). Use and store the insulin in compliance with the manufacturer specifications. Check the use by date of the insulin before use.

#### Warning and Filling the Reservoir

#### WARNING

##### Risk of hypoglycaemia (low blood glucose level) and hyperglycaemia (high blood glucose level)

The micropump system is intended for use with U100 insulin. Use U100 insulin only.

To prepare the reservoir, you need a rapid-acting U100 insulin and disinfectant, such as a sterile alcohol wipe. Ensure that the reservoir and the insulin are at room temperature during filling. To replace a component of the micropump system, always use the *Replace* menu on the diabetes manager.

To view an animated video of the handling steps, tap *Help* on the diabetes manager.

- 1 Wash your hands and dry them thoroughly. Disinfect the top of the insulin vial with a sterile alcohol wipe. Allow the disinfected top of the insulin vial to dry.
- 2 Slowly and carefully pull out the battery's protective film (D) downwards in the direction of the arrow to activate the battery.
- 3 Hold the round part of the handle (C) firmly and pull it downwards in the direction of the arrow to fill the reservoir (A) with air. Fill the reservoir with the volume of air that you later want to fill with insulin.
- 4 Place the insulin vial on a flat and solid surface (for example, a table top) and hold it firmly. Place the filling aid (B) onto the vial. Push the filling aid downwards until you hear a click into place. Press the handle all the way down in the direction of the arrow to fill the insulin vial with air.
- 5 Turn the reservoir assembly together with the insulin vial upside down so that the vial is above the reservoir.
- 6 Slowly pull the handle downwards in the direction of the arrow until the reservoir with insulin. Try to ensure that as few air bubbles as possible form in the reservoir. Fill the reservoir with at least 80 U (0.8 ml) of insulin but with no more than 200 U (2.0 ml) of insulin.
- 7 Check the reservoir from several angles to make sure that there are a few air bubbles as possible in the reservoir. To release air bubbles from the reservoir, hold the reservoir at an angle. Gently flick your finger against the reservoir several times.
- 8 Slowly push the handle upwards in the direction of the arrow
  - to remove air bubbles from the reservoir. Slowly pull the handle downwards in the direction of the arrow
  - until the reservoir is filled with the required amount of insulin again.An air bubble with the size of the number 0 printed on the reservoir scale corresponds to an insulin quantity of 0.4 U. Air bubbles can cause incorrect insulin delivery.
- 9 Remove the filling aid from the reservoir by pulling it off laterally in the direction of the arrow. Compress the handle on the ribbed surface ➊ and then remove the handle laterally on the piston rod (F) ➋.
- 10 Align the piston rod (F) of the filled reservoir such that you can insert it into the piston rod opening of the pump base. Push the filled reservoir onto the pump base until both parts are tightly connected.
- 11 When both components are connected to each other correctly and the battery is activated, the micropump issues the *Start* sound. If this sound is not issued, check whether the battery is activated and repeat step 10.
- 12 Make sure that there is no gap between the reservoir and the pump base. The pump base and the reservoir are only connected correctly when the *Start* sound is issued and when the pump base and the reservoir are seamlessly connected.

#### Disposal

#### WARNING

##### Risk of infection

Used components carry a risk of infection. Dispose of the reservoir assembly, the filling aid, and the handle for the piston rod safely, in such a way that no third party can injure themselves.

Dispose of used products according to local regulations. For information about correct disposal, contact your local council or authority.

#### Distributed in the United Kingdom by:

Roche Diabetes Care Limited  
Charles Avenue, Burgess Hill  
West Sussex, RH15 9RY, United Kingdom  
Accu-Chek Pump Careline ™  
UK Freephone number: 0800 731 22 91  
RD Freephone number: 1 800 89 23 51  
\*calls may be recorded for training purposes

Some mobile operators may charge for calls to these numbers. Burgesshill.insulinpumps@roche.com  
www.accu-check.co.uk, www.accu-check.ie

#### Australia

Roche Diabetes Care Australia Pty. Limited  
Insulin Pump Support: 1800 633 457  
australia.insulinpumps@roche.com  
www.accu-check.com.au

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