CE 0123

ACCU-CHEK, ACCU-CHEK RAPID-D LINK and AKKY-CHEK are trademarks of Roche.

© 2019 Roche Diabetes Care

Made in Poland

Roche Diabetes Care GmbH  
Sandhofer Strasse 116  
68305 Mannheim, Germany  
[www.accu-chek.com](http://www.accu-chek.com)

Last update: 2019-04

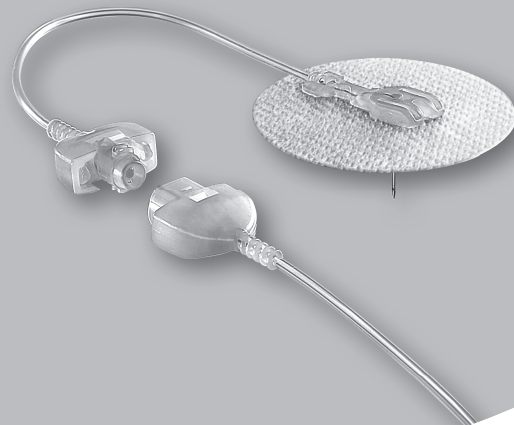
04537220001/N

**ACCU-CHEK® Rapid-D Link**









## Instructions for use

Infusion set

**ACCU-CHEK®**

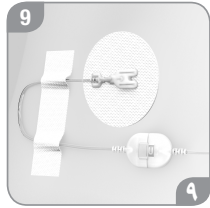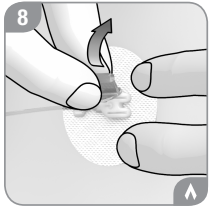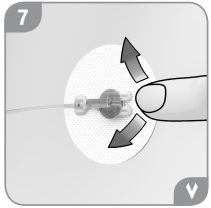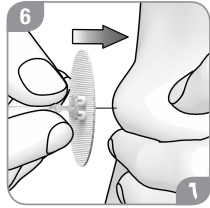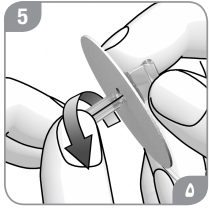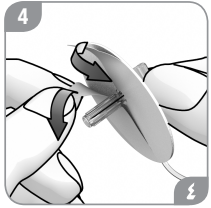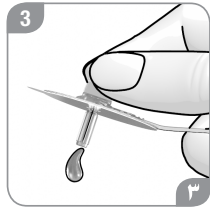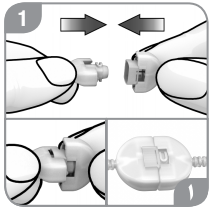
<b>Instructions for use</b>	<b>4</b>	<b>Návod k použití</b>	<b>116</b>
<b>Gebrauchsanweisung</b>	<b>12</b>	<b>Návod na použitie</b>	<b>124</b>
<b>Instructions d'utilisation</b>	<b>20</b>	<b>הוראות שימוש</b>	<b>139</b>
<b>Istruzioni per l'uso</b>	<b>30</b>	<b>Használati útmutató</b>	<b>140</b>
<b>Instrucciones de uso</b>	<b>40</b>	<b>Navodila za uporabo</b>	<b>148</b>
<b>Instruções de utilização</b>	<b>48</b>	<b>Instrucțiuni de utilizare</b>	<b>156</b>
<b>Gebruiksaanwijzing</b>	<b>56</b>	<b>Инструкции по использованию</b>	<b>164</b>
<b>Brugsanvisning</b>	<b>64</b>	<b>Інструкція з використання</b>	<b>176</b>
<b>Bruksanvisning</b>	<b>72</b>	<b>Kasutusjuhend</b>	<b>184</b>
<b>Bruksanvisning</b>	<b>80</b>	<b>Lietošanas pamācība</b>	<b>192</b>
<b>Käyttöohje</b>	<b>88</b>	<b>Naudojimo taisyklės</b>	<b>200</b>
<b>Οδηγίες χρήσης</b>	<b>96</b>	<b>دليل الاستخدام</b>	<b>217</b>
<b>Instrukcja obsługi</b>	<b>106</b>		

<b>Instructions for use</b>	<b>en</b>
<b>Gebrauchsanweisung</b>	<b>de</b>
<b>Instructions d'utilisation</b>	<b>fr</b>
<b>Istruzioni per l'uso</b>	<b>it</b>
<b>Instrucciones de uso</b>	<b>es</b>
<b>Instruções de utilização</b>	<b>pt</b>
<b>Gebruiksaanwijzing</b>	<b>nl</b>
<b>Brugsanvisning</b>	<b>da</b>
<b>Bruksanvisning</b>	<b>no</b>
<b>Bruksanvisning</b>	<b>sv</b>
<b>Käyttöohje</b>	<b>fi</b>
<b>Οδηγίες χρήσης</b>	<b>el</b>
<b>Instrukcja obsługi</b>	<b>pl</b>


# ACCU-CHEK® Rapid-D Link
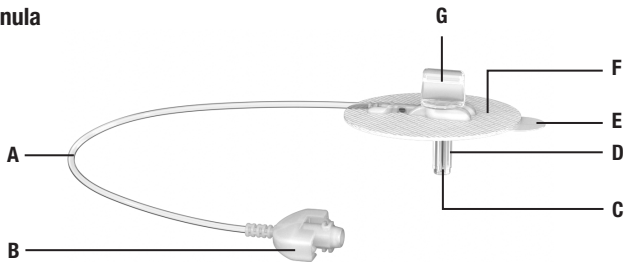
## Infusion set

### Cannula

### Transfer set
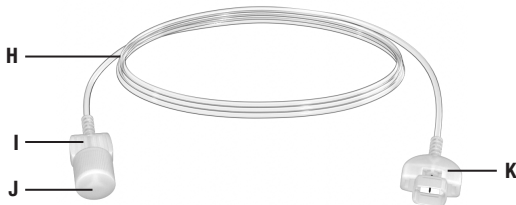
### Protective cap

## Description

### Cannula

- A** Tubing
- B** Connector
- C** Steel cannula
- D** Needle cover
- E** Protective film
- F** Adhesive pad

### Transfer set

- H** Tubing
- I** Luer-lock connector
- J** Protective cap
- K** Connector

### Protective cap

- L** Protective cap

## Contents of the pack

There are three packages with different contents. The cannula, transfer set and protective cap are packed in their own separate packages.

## Intended use

The Accu-Chek Rapid-D Link infusion set is intended for the subcutaneous infusion of insulin administered with an insulin pump.

These instructions for use feature the following symbol:

This symbol draws your attention to a warning. A warning must be heeded because it indicates a risk of injury or of damage to your health or the health of others. Not heeding the warnings can lead to life-threatening situations.

## General warnings

- Read these instructions for use carefully before using the infusion set. Also read the information about insulin pump therapy contained in the User's Manual of the insulin pump.
  - When you use the infusion set for the first time, your healthcare team must be present.
  - Prepare the infusion site as instructed by your healthcare professional before you insert the cannula.
  - This product is sterile packaged. Do not use the product if the package has already been opened or is damaged.
  - Protect the product from excessive humidity, sunlight and heat. Store the product at room temperature.
  - Do not allow the infusion set to come into contact with disinfectants, perfumes, deodorants or insect repellents.
  - Dispose of the used product according to local regulations. Do not clean or sterilise the used product.
  - Avoid mechanical stress on the infusion set and infusion site.
- The infusion set contains small parts. There is a risk of suffocation if small parts are swallowed. Keep all parts of the infusion set away from persons who may not be aware of the risks and may put small parts in their mouth, for example, small children.
  - Pay attention to the tubing of the infusion set connected to your body. There is a risk of strangulation if the tubing becomes wrapped around your neck. In case of doubt, use the shortest tubing length to minimise the risk of strangulation.

## Product-specific warnings

- Change the cannula at least every 2 days and the transfer set at least every 6 days. Otherwise, there is a risk that the infusion site may become inflamed or the insulin may not be as effective.
- If the infusion site is inflamed, replace the infusion set immediately and choose another infusion site.
- Check your blood glucose level at least four times a day and at least once within 1 to 3 hours of replacing the transfer set or cannula.
- Do not replace the cannula or transfer set immediately before going to bed.
- If your blood glucose level should unexpectedly increase, or if an occlusion alarm occurs, check the infusion set for occlusions or leaks. If you are not sure whether the infusion set is working properly, replace it immediately.
- Never fill the tubing or attempt to remove an occlusion in the tubing while the infusion set is connected to your body as you may deliver an uncontrolled quantity of insulin.

- Check the connection between the luer-lock connector and the tubing every 3 hours and before you go to bed. If the connection is damaged, replace the transfer set immediately.
- If the cannula was pulled out of the skin, you must insert a new cannula.

## Attaching the infusion set

- ▶ Choose an infusion site away from your waistline, any scar tissue, bones, navel and recent infusion sites.
- ▶ Wash your hands.
- ▶ Disinfect the chosen infusion site as instructed by your healthcare team.
- ▶ Let your skin dry completely.

**1**

- ▶ Hold both connectors (B and K) so that they form a cross.
- ▶ Plug the connectors into each other.
- ▶ Turn both connectors a quarter turn until you hear and feel them click into place. You can only turn the connectors in one direction.

If you do not hear and feel both connectors click into place, replace the cannula and the transfer set.

**2**

- ▶ Unscrew the protective cap (J) from the luer-lock connector (I).
- ▶ Screw the luer-lock connector (I) into the insulin pump adapter.

Do not use any tools to screw the luer-lock connector into the insulin pump adapter. This could damage the luer-lock connector.

**3**

- ▶ Fill the infusion set as described in the User's Manual of the insulin pump until insulin drips from the steel cannula (C). There must not be any air bubbles in the cartridge or infusion set.

### Fill volume of the infusion set (U100 insulin)

Length	Units	Volume
30 cm / 12"	approx. 7 U	approx. 70 µL
60 cm / 24"	approx. 10 U	approx. 100 µL
80 cm / 31"	approx. 13 U	approx. 130 µL
110 cm / 43"	approx. 18 U	approx. 180 µL

You must fill up the infusion set (tubing and cannula) completely before inserting the cannula into the skin. The infusion set must not contain any air bubbles.

4

- ▶ Hold the cannula by the handling aid (G).
- ▶ Remove both parts of the protective film (E) from the adhesive pad (F).

5

- ▶ Unscrew the needle cover (D) to remove it.

6

- ▶ Form a skin fold at the chosen infusion site.
- ▶ Insert the steel cannula (C) quickly and vertically (at a 90° angle) into the skin fold.

7

- ▶ Press the adhesive pad (F) onto the skin.

8

- ▶ Hold the adhesive pad (F) in place.
- ▶ Bend the handling aid (G) to the side until it detaches itself.

9

- ▶ Arrange the tubing (A) to form a crescent.
- ▶ Use an adhesive pad to tape the crescent

to the skin. The crescent prevents the weight of both connectors from pulling on the cannula.

## Connecting and disconnecting the insulin pump

This infusion set allows you to disconnect and remove the insulin pump for showering and swimming, for example. You can reconnect the insulin pump without having to replace the infusion set.

Consult your healthcare team on how to compensate for any missed insulin while you are disconnected from the insulin pump.

### Disconnecting the insulin pump

- ▶ Turn both connectors a quarter turn in opposite directions until they form a cross. You can only turn the connectors in one direction.
- ▶ Pull the connectors apart.
- ▶ Place a protective cap (L) on each connector.

When you disconnect the pump, place a protective cap on each connector.

## Connecting the insulin pump

- ▶ Remove the protective caps from the connectors.
- ▶ Connect the tubing (H) and the cannula using the connectors as described in **1**.

## Replacing the cannula

- ▶ Disconnect the pump.
- ▶ Carefully pull off the adhesive pad (F), working round the outside edge into the centre.
- ▶ Remove the steel cannula (C) from the skin in a vertical manner and completely remove the adhesive pad simultaneously.

Dispose of the used cannula in such a way that it will not cause injury to anyone. Place the cannula in a cannula disposal container, for example.

- ▶ Connect the pump to a new cannula as described in step **1**.
- ▶ Then continue with step **3**.

## Replacing the transfer set

- ▶ Disconnect the pump.
- ▶ Unscrew the luer-lock connector (I) from the insulin pump adapter.
- ▶ Connect a new transfer set to the pump as described in step **2**.
- ▶ Fill the new transfer set as described in the User's Manual of the insulin pump.
- ▶ Connect a new transfer set to the cannula as described in step **1**.

## Customer support

### Australia

Roche Diabetes Care Australia Pty. Limited  
Pump Support: 1800 633 457  
[www.accu-chek.com.au](http://www.accu-chek.com.au)

### New Zealand

Accu-Chek Enquiry Line: 0800 80 22 99  
Pump Hotline: 0800 696 696  
[www.accu-chek.co.nz](http://www.accu-chek.co.nz)

Roche Diabetes Care South Africa (Pty) Ltd.  
Hertford Office Park, 90 Bekker Road  
Vorna Valley, Midrand

### South Africa

1686  
Email: [info@accu-chek.co.za](mailto:info@accu-chek.co.za)  
Call Toll Free: 080-34-22-38-37 (SA only);  
+254 20 764 0560 (Kenya only);  
+27 (11) 504 4677 (Other countries)

### Singapore

Accu-Chek ExtraCare line: 6272 9200  
[www.accu-chek.com.sg](http://www.accu-chek.com.sg)

### Distributed in the United Kingdom by:

Roche Diabetes Care Limited  
Charles Avenue, Burgess Hill  
West Sussex, RH15 9RY, **United Kingdom**

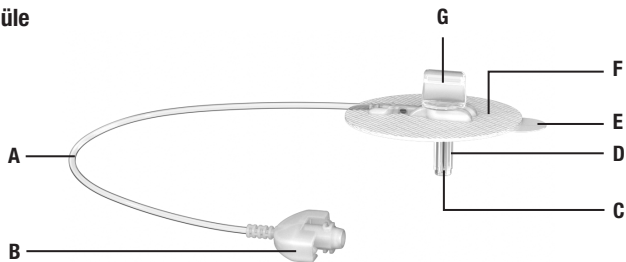
Accu-Chek Pump Careline <sup>1)</sup>:  
UK Freephone number: 0800 731 22 91  
ROI Freephone number: 1 800 88 23 51

<sup>1)</sup> calls may be recorded for training purposes

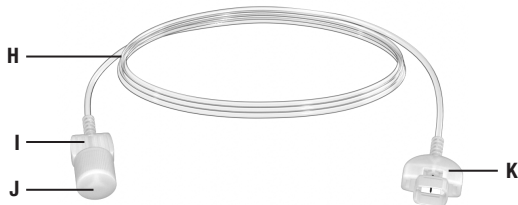
Some mobile operators may charge for calls to these numbers.  
[burgesshill.insulinpumps@roche.com](mailto:burgesshill.insulinpumps@roche.com)  
[www.accu-chek.co.uk](http://www.accu-chek.co.uk)  
[www.accu-chek.ie](http://www.accu-chek.ie)

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